

SUGGESTED PROCEDURE

- Attach the filter's tubing adapter to the suction (vacuum) port to the collection canister lid.
- Attach the other filter port to the vacuum source.



WARNING

- Connection of the tubing adapter to the other non-rigid tubing could cause obstruction of the tubing resulting in premature canister shut-off.



CAUTION

- For single patient use only. Reuse can cause cross infections. Use of sterilization or cleaning solution may dangerously increase resistance and/or result in retention of harmful residuals.
- Federal law in the U.S.A. and Canada restricts this device to sale by, or on the order of, a licensed physician only.

Reorder Item Number:

5787001
(package/20)